

Iraq Ministry of health

November 15, 2023

DHPC: Simponi (golimumab) 50 mg and 100 mg: Important changes to the injection instructions for the SmartJect Pre-filled Pen.

Dear Healthcare Professional,

The marketing authorization holder of Simponi, Janssen Biologics B.V., and the local representative, Shubber team in agreement with Iraqi Pharmacovigilance Center would like to inform you of the following:

Summary

- **Accidental needle stick injuries, bent or hooked needles, and device actuation failure have been reported for the Simponi SmartJect pre-filled pen.**
- **Instructions for use have therefore been revised as follows:**
 - **Do not put the cap of the pre-filled pen back if removed, to avoid bending the needle.**
 - **Only inject in the thigh or abdomen.**
 - **Use a two-hand approach to administer the injection (one hand to hold the pre-filled pen and the other hand to press the blue button to start the injection).**
 - **Do not pinch the skin, when positioning the pre-filled pen and when administering the injection.**
- **The device must be pushed against the skin until the green safety sleeve slides completely into the transparent cover BEFORE the blue button is pressed. Only the wider portion of the green safety sleeve remains outside of the transparent cover.**
- **All patients/caregivers, including those previously trained on the SmartJect pre-filled pen, should be instructed on the proper use of the device in accordance with the revised instructions for use.**

Background of the safety concern

SIMPONI is available as a solution for monthly subcutaneous administration. In the EU, more than one delivery device presentation is available (SmartJect pre-filled pen and Simponi pre-filled syringe). This safety communication concerns the SmartJect pre-filled pen only.

As per the approved product information, after proper training in subcutaneous injection technique, patients may self-inject if their physician determines this is appropriate, with retraining as necessary. Patients should be instructed to inject the prescribed amount of Simponi according to the comprehensive instructions for use (IFU) provided in the package leaflet.

An investigation of product complaints and adverse events related to the SmartJect pre-filled pen identified the following problems:

Johnson & Johnson Middle East FZ-LLC Lebanon Branch
Sin El Fil, General Chehab Avenue, Paul Haddouche Building, Near Clinique du Levant
P.O.Box 55-325 Beirut LEBANON
Phone +961 1 518700 – Fax +961 1 518797

The information provided herein contains Company trade secrets, commercial or financial information that the Company customarily holds close and treats as confidential. The information is being provided under the assurance that the recipient will maintain the confidentiality of the information under applicable statutes, regulations, rules, protective orders or otherwise.

- Accidental needle stick injuries to the healthcare provider or caregiver when pinching the skin during the injection;
- Bent or hooked needles that may require medical/surgical intervention to remove the needle from the injection site, most commonly occurring with arm injections;
- Inability to depress the pre-filled pen button and initiate the injection due to users pressing the button prematurely.

Accordingly, the SmartJect IFU, which is located within the package leaflet in the product package, has been revised. This safety communication is intended to inform you about the revised IFU.

Highlights of the revised IFU:

- Do not put the cap of the pre-filled pen back if removed to avoid bending the needle.
- The front of the thigh or the lower abdomen should be used as injection sites. **The arm should not be used as an injection site for the SmartJect pre-filled pen.**
- The pre-filled pen should be held comfortably with one hand, above the blue button, to avoid touching or pressing the button prematurely.
- The open end of the pre-filled pen should be pushed straight against the skin at a 90-degree angle to slide the green safety sleeve inside the clear cover. The blue button should not be pressed until after the green safety sleeve has completely slid into the transparent cover. Only the wider portion of the green safety sleeve remains outside of the transparent cover.
- **The skin should not be pinched** when positioning the pre-filled pen flat against the skin or when administering the injection.
- The hand not holding the pre-filled pen should be used to press the blue button to start the injection.
- The **sequence of steps described in the IFU must be followed** to ensure proper actuation of the device for injection.

Requested action:

- All patients/caregivers should be informed on the proper use of the pre-filled pen in accordance with the revised IFU. This would include those who were previously educated in using the prior IFU.
- This communication should be shared with personnel involved in educating patients and/or their caregivers on the SmartJect pre-filled pen.

Call for reporting

Healthcare professionals should report any suspected adverse reactions associated with the use of SIMPONI in accordance with the national spontaneous reporting system

Johnson & Johnson Near East Pharmacovigilance department	Iraqi Ministry of Health
---	--------------------------

<p>Email: PVNEMA@its.jnj.com Phone: 00961 81 312 867</p>	<p>@Iraqi Pharmacovigilance Center Ministry of Health, DTA Bab Al-Mu'adham, Baghdad, Iraq Phone +9647807820490 Email: iqphvc@yahoo.com, iraqiphvc@moh.gov.iq</p> <p>E- reporting link address: https://primaryreporting.who-umc.org/IQ</p> 
---	---

Company contact points

Johnson & Johnson Middle East FZ-LLC

Lebanon Branch

Sin El Fil, General Chehab Avenue

Paul Habbouche Building

Near Clinique du Levant

PO Box 55-325 Beirut, LEBANON

Phone +961 1 518700

Summow Shubber General Trading & Commercial Agencies Co Ltd

Address: Building 10, Street No 17, District No 101, Bataween, Baghdad, Iraq

Email: mohammed@alshubbergroup.com

Phone number: 009647841592909

Johnson & Johnson Middle East FZ-LLC Lebanon Branch
 Sin El Fil, General Chehab Avenue, Paul Haddouche Building, Near Clinique du Levant
 P.O.Box 55-325 Beirut LEBANON
 Phone +961 1 518700 – Fax +961 1 518797

The information provided herein contains Company trade secrets, commercial or financial information that the Company customarily holds close and treats as confidential. The information is being provided under the assurance that the recipient will maintain the confidentiality of the information under applicable statutes, regulations, rules, protective orders or otherwise.

Yours sincerely,

Bertha NAOUM

Country Medical Head

Levant&Iran